

limited lifetime warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

exam lamp

item #13408



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

A large, vertical version of the Drive logo, with the word "drive" in white lowercase letters on a dark blue background. The 'i' has a yellow and orange square above it. A small "TM" trademark symbol is located at the top right of the word.

limited lifetime warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

exam lamp

item #13408MB



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

A large, vertical version of the Drive logo, with the word "drive" in white lowercase letters on a dark blue background. A small yellow and orange square is positioned above the letter 'i'. A trademark symbol (TM) is located at the top right of the word.

assembly instructions

WARNING:

BULB IS NOT TO EXCEED A MAXIMUM OF 60 WATTS.

Set-Up Standard pole


1. Thread pole (A) onto adapter on base (C).
2. Thread Flex arm (B) onto pole set.

WARNING:

The lamp is equipped with a 3-prong grounding plug for protection against shock hazards. DO NOT under any circumstances cut or remove the grounding prong from the plug. The plug must be used directly with the correct 3-prong receptacle. DO NOT use an extension cord or adapter. To make alterations call a professional electrician.



REF. 13408MB Shown

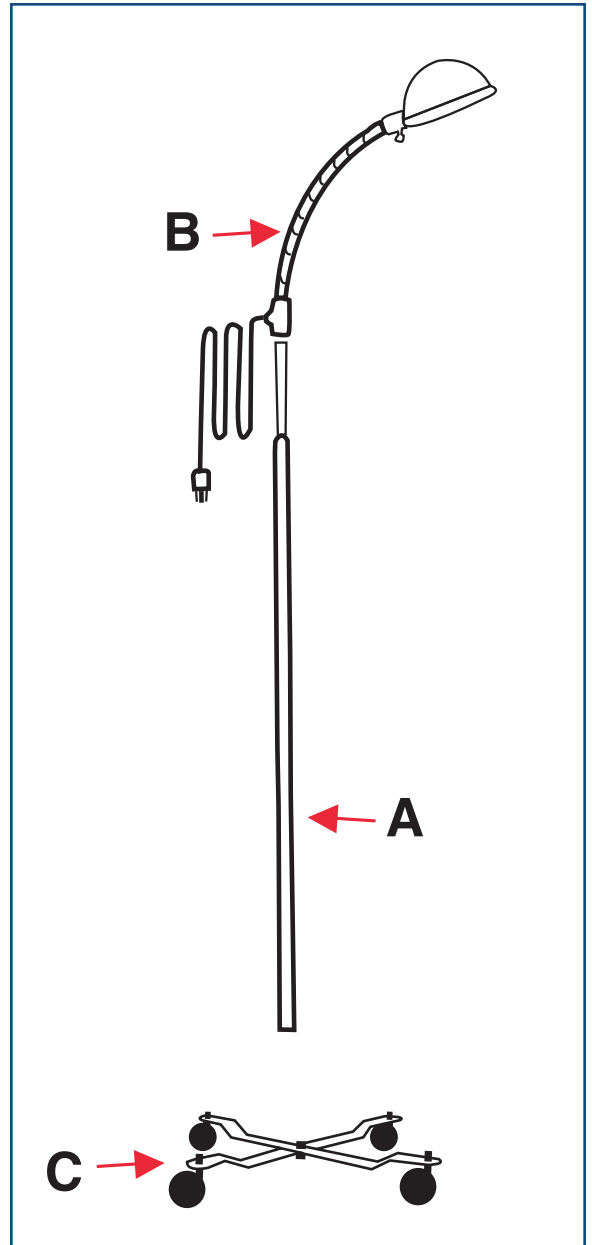

assembly instructions

WARNING:

BULB IS NOT TO EXCEED A MAXIMUM OF 60 WATTS.

Set-Up Standard pole

1. Thread pole (A) onto adapter on base (C).
2. Thread Flex arm (B) onto pole set.

WARNING:

The lamp is equipped with a 3-prong grounding plug for protection against shock hazards. DO NOT under any circumstances cut or remove the grounding prong from the plug. The plug must be used directly with the correct 3-prong receptacle. DO NOT use an extension cord or adapter. To make alterations call a professional electrician.

