drive™

[www.drivemedical.com](http://www.drivemedical.com)

Rev1.05.08.14

stackable bath step

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drive™

[www.drivemedical.com](http://www.drivemedical.com)

The Drive Bath Step is designed to make getting in and out of the bath or tub easier. The Drive Bath Step features a large flat slip proof stepping surface with built-in non-slip pads. Two or more steps can be combined to raise the height of the steps by following the instructions below.

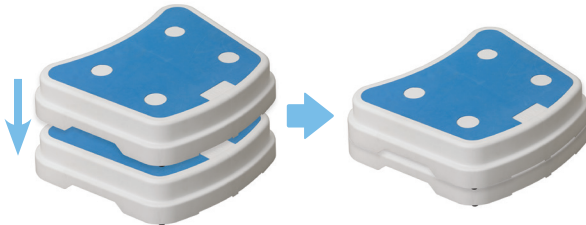
## instructions

To combine bath steps, follow the instructions below.

1. Place bath step on flat surface.
2. Using a coin, remove each of the 4 white circles located on the top of the bath step. Store these for future use.

3. Place second bath step on top of first, lining up prongs on underside of step with receptacle holes. Press down firmly to secure in place.

## 1-year limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year of the original date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This 1 year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

The logo for Drive Medical, featuring the word "drive" in a bold, blue, lowercase sans-serif font. A small yellow square is positioned above the letter "i". A trademark symbol (TM) is located at the top right of the logo.

© 2014 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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©2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.