


LIGHTWEIGHT EXPEDITION TRANSPORT CHAIR (EXP19LT)

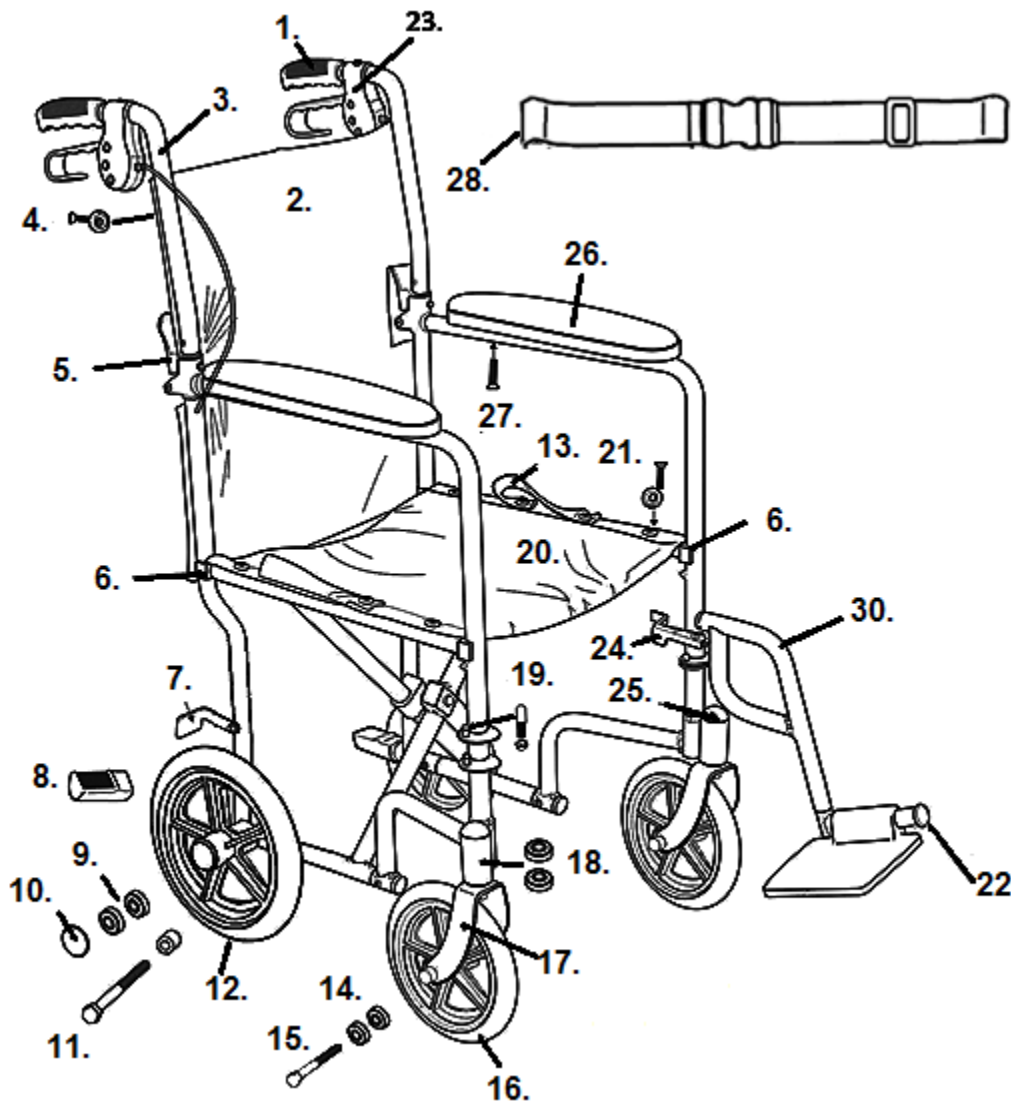

- | | |
|-------------------------------------------------|-------------------------------------------------|
| 1. Handgrip- STDS871 | 16. Caster- STDS5D5515 |
| 2. Back Upholstery- STDS5D5501 | 17. Fork Blue- STDS5D5516 |
| 3. Push Handle Blue- STDS5D5502 | Fork Red- STDS5D5517 |
| Push Handle Red- STDS5D5503 | 18. Fork Stem Bearings (2 ea)- STDS5D5518 |
| 4. Back Upholstery Hardware (10 ea)- STDS5D5504 | 19. Hanger Pins (4 ea)- STDS5D1069 |
| 5. Folding Cane Hinge- STDS5D0914 | 20. Seat Upholstery- STDS3J0929 |
| 6. Seat Rail Guide- STDS1070 | 21. Seat Upholstery Hardware (8 ea)- STDS5D5519 |
| 7. Wheel Brake Left- STDS5D5505 | 22. Footrest Bumper- STDS1026 |
| Wheel Brake Right- STDS5D5506 | 23. Hand Brake & Cable- STDS5D5520 |
| 8. Tipping Sleeve- STDS4020 | 24. Release Lever Left- STDS3J091L |
| 9. Rear Wheel Bearings (2 ea)- STDS5D5507 | Release Lever Right- STDS3J091R |
| 10. Hub Cap- STDS5D5508 | 25. Dust Cap- STDS1051 |
| 11. Rear Axle & Sleeve- STDS5D5509 | 26. Armrest Pad- STDSFLVB |
| 12. Rear Wheel - STDS5D5510 | 27. Arm Screw- STDS002001 |
| 13. Upholstery Strap- STDS3J4504 | 28. Seat Belt- STDS850-PB |
| 14. Caster Bearings (2 ea)- STDS5D5513 | 29. Footrest Blue (pr)- EXPLTSF-BL |
| 15. Caster Axle- STDS5D5514 | Footrest Red (pr)- EXPLTSF-RD |